



# claris i310D

## USB Cable Connector Guide



1. Please find the USB Cable connector for your Claris Intraoral Camera.

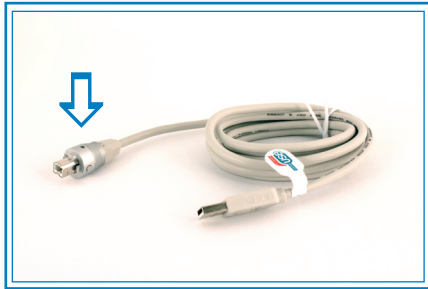



2. Firmly press the the two buttons on the opposing sides to release the inner USB latch.

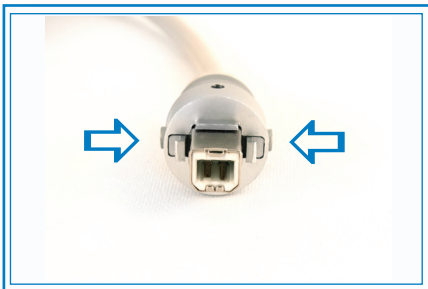

3. While pushing the two buttons, connect the tip of the USB connector into the back end of the Claris camera and snap into place.

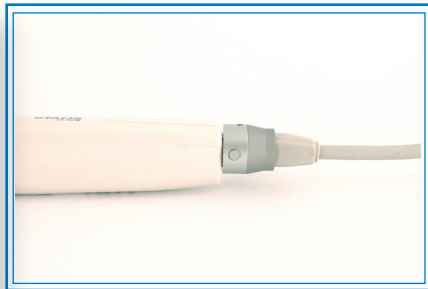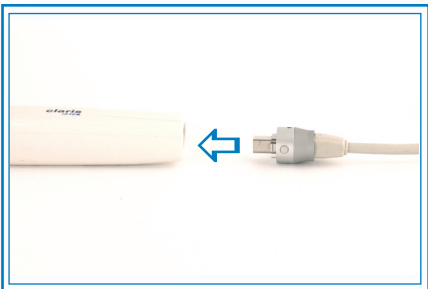

4. To disconnect the USB cable, press the two buttons, gently wiggle the cable, and pull to disconnect from the USB.

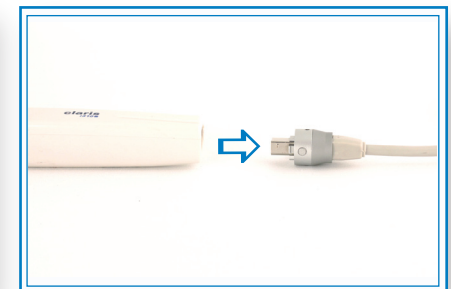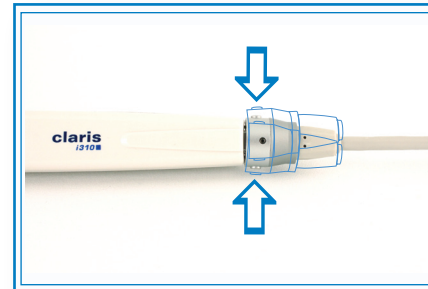

sota  
IMAGING

tel: 1.888.266.SOTA  
fax: 1.714.532.6107  
www.sotaimaging.com

1073 n. batavia st. suite b, orange, ca 92867